

SOMNI  
SCIENTIFIC



# THE SOMNI WORK STATION ASSEMBLY GUIDE

## ► EQUIPMENT NEEDED:

4 mm Allen wrench


Pre-assembled base and screws inserted into the holes they are designed for.




1

### STEP 1:

Remove base from the shipping box. The concentrator plate and wheels will be pre-assembled with the back support bracket to the base plate frame.

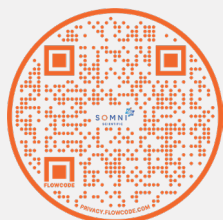


(T) 0800 0129101 (D) 01872 248890 (M) 07798 969805

enquiries@somniscientific.com  www.somniscientific.co.uk

2

## STEP 2:


Stabilize the middle shelf, insert into riser post. Fasten the three screws in the riser post.

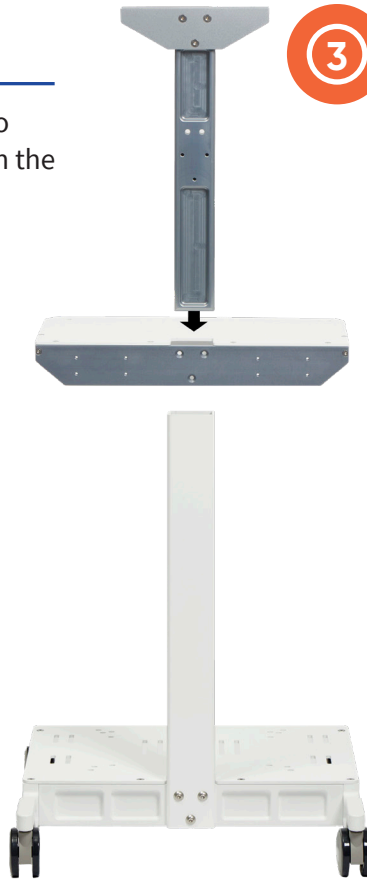

3

## STEP 3:

Insert riser post into back of support bracket. Fasten 2 screws on each side.

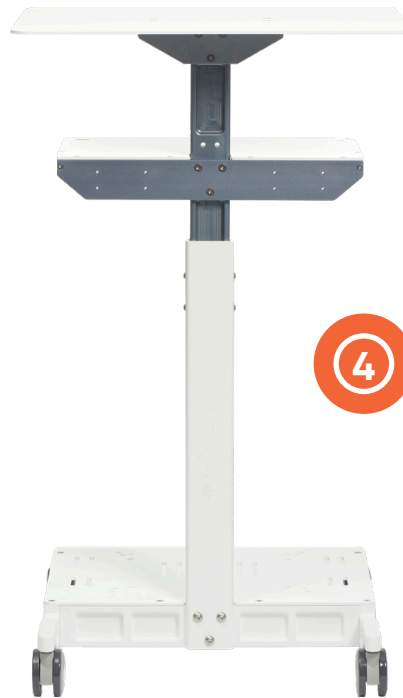


4

## STEP 4:

Place top shelf on supporting brackets of the center beam and screw in.



(T) 0800 0129101 (D) 01872 248890 (M) 07798 969805

enquiries@somniscientific.com  www.somniscientific.co.uk